


CANCER GENETICS ITALIA 13 18 21 PROBE MARKETED BY KAMINENI LIFE SCIENCES
KIT INSERT FOR SAMPLES

13-18-21 FISH Probe
Three Color Copy Number Probe
Ref: **13-18-21**

Instructions for Use

Intended use

The **13-18-21** FISH Probe is designed to detect changes in copy number of the chromosomes **13, 18 and 21** by Fluorescence In Situ Hybridisation (FISH). Increased copy number of each of these chromosomes has been observed as a constitutional trisomy in specimens of pregnancy loss and live born infants.

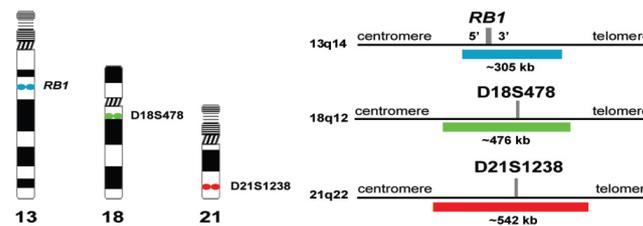

Schematic representation of the 13/18/21 (***RB1/D18S478/D21S1238***) FISH Probe:

Horizontal aqua, green, and red bars indicate the regions covered by the probes. The directly labeled ***RB1***(aqua), ***D18S478***(green), and ***D21S1238*** (red) probes span the respective loci.

Storage

- Storage of FISH Probe: Store at -20°C protected from light until the expiry date as indicated on the label.
- Storage of Slides: Store hybridized slides at -20°C protected from direct light.
Note: The storage conditions apply to both opened and unopened products; vials stored under other conditions may not perform optimally and will affect the assay result. The number of freeze/thaw cycles should not exceed the recommended number of tests per vial. Store in original container.

Handling

- Handle all reagents as capable of transmitting infectious agents and dispose of according to current national law.
- Handle all reagents and slides containing fluorophores in reduced light to prevent photobleaching.

Warnings and Precautions

Please read instructions before proceeding.

- Improper transportation or storage can destroy or impair the performance of the product.
- If any of the following occurs: receipt of compromised package or vial, device failure (when used according to instructions for use), or injury of user, please contact the manufacturer.
- Any compromised vial should be discarded according to current national law and assays should not be performed from such reagent.
- Handle all reagents with care and wear appropriate personal protective equipment.
- Formamide, Saline-Sodium Citrate (SSC), and Sodium Dodecyl Sulfate (SDS) may have teratogenic and mutagenic effects: avoid inhalation, ingestion or contact with skin.
- DAPI is an irritant. See Material Safety Data Sheet (MSDS) for safety information.



Reagent provided

Ready-to-use FISH Probe: 100µl per vial (10 tests); one test is defined as sufficient for a 22 x 22 mm area. The FISH Probe is premixed in hybridization buffer (formamide, dextran sulphate, and SSC) and contains directly labelled probes for ***RBI*** loci(aqua), **D18S478** loci(green), and **D21S1238** loci (red).

Reagent, Material, and Equipment required but not provided

Equipment:

Coplin jars
Coverslip (22x22 mm & 25x25 mm)
Epi-fluorescent microscope with appropriate filters
Forceps
Fume hood
Gloves
Humidified chamber
Immersion oil
Incubator
Mercury lamp (100 watt)
Microcentrifuge
Microfuge tube (0.5 mL)
Micropipette (1-200 mL)
pH meter
Rubber cement
Slide tray
Slide warmer
Thermometer, calibrated
(37°C to 80°C)
Hot plate
Water bath

Reagents:

100% Ethanol
10X PBS
1N HCl
1M MgCl₂
1M NaOH
20X SSC
10% Formalin
DAPI/Antifade
Distilled Water
Pepsin
Tween 20

Reagent preparation

Note: Use distilled water for the preparation of all stock and working solutions.

- Ethanol Series (70%, 85%, and 100%): Prepare v/v dilutions of 100% ethanol with distilled water (dH₂O). Store at RT.
- 0.01N HCl (Hydrochloric acid): Add 0.5 mL of 1N HCl to 49.5 mL (dH₂O) Store at RT. Pre-warm the solution to 37°C in a waterbath.
- 0.4% (4 mg/mL) Pepsin Stock Solution: Dissolve 100 mg of pepsin in 25 mL 0.2N HCl. Store 500 mL aliquots at -20°C.
- 1% Formaldehyde: Add 12.5 mL of 10% formalin (4% formaldehyde) to 37 mL of 1X PBS. Add 500 mL of 100X MgCl₂. Store up to 1 week at 4°C.
- 0.5X SSC (Saline-Sodium Citrate)/0.1% Tween 20: Add 25 mL of 20X SSC and 1 mL of Tween 20 to 974 mL dH₂O. Mix well by swirling. Store at RT
- 1X PBS (Phosphate Buffer Saline): Mix 100 mL of 10X PBS to 900 mL dH₂O. Adjust pH to 7.0. Store at RT.
- 2X SSC: Mix 100 mL of 20X SSC and 900 mL of dH₂O. Adjust pH to 7.0. Store at room temperature (RT).
- 2X SSC/0.1% Tween 20: Add 100 mL of 20X SSC and 1 mL of Tween 20 to 899 mL dH₂O. Mix well by swirling. Store at RT.
- 100X MgCl₂ (Magnesium Chloride) in 1X PBS: Add 50 µL of 1M MgCl₂ to 450 µL of 1X PBS.

Standard FISH Procedure

Note: Products ready-to-use. Do not reconstitute or dilute with hybridization buffer. For professional use only.

- Only a technologist familiar with cytogenetic methods and trained for the FISH technique can perform the assay. All equipment should be calibrated prior to performance of the assay.
- The intended tissue is Amniotic Fluid, Chorionic villus sampling (CVS) & Cord blood. The slides should be prepared according to the guidelines for standard cytogenetic methods of the laboratory performing the assay.

Slide Preparation

All freshly prepared slides should be aged for 1.5 hours at 45°C before hybridization. If not hybridizing same day as prepared, store at +4°C or -20°C for long-term storage. Slides with visible cytoplasm may require pretreatment with proteolytic enzyme (see optional pretreatment with pepsin).

Optional: Slide Pretreatment with Pepsin

1. Pre-warm 50 mL of 0.01N HCl at 37°C.
2. Add 25 µL of 0.4% pepsin stock to the pre-warmed 50 mL of 0.01N HCl and incubate the slide for 5 min at 37°C in pepsin.
Procedure Note: Some specimens may require longer digestion time in pepsin or a higher concentration of pepsin.
3. Wash the slide twice for 5 min in 1X PBS at room temperature (RT).
4. Incubate the slide for 5 min in 1% formaldehyde at RT.
5. Wash the slide twice for 5 min in 1X PBS at RT.
6. Dehydrate the slide in 70%, 85%, and 100% ethanol at RT for 1 min each.
7. Air dry the slide.

Procedure Note: Check the morphology of the sample with a phase contrast micro-scope before hybridization. Do not hybridize if the nuclear morphology is compromised.

FISH Probe Denaturation / Hybridization

1. Vortex the FISH Probe briefly and spin the tube in a microcentrifuge.
2. Apply 10 μ L of the FISH Probe to the target area and cover with a coverslip (22x22 mm).
Procedure Note: Care should be taken to avoid air bubbles. Smaller or larger cover-slips may be used with proportional change in FISH probe volume.
3. Seal the edges of the coverslip thoroughly with rubber cement.
4. Co-denature the slide and the FISH Probe for 3 min at 80°C on a temperature controlled hot plate or an automated hybridization device.
5. Incubate for 12-18 hours in a humidified environment at 37°C protected from direct light.

Standard FISH Procedure

Post Hybridization Washing

Procedure Note: Do not allow the slide to dry before washes are complete.

1. Pre-warm the 2X SSC/0.1% Tween 20 & 0.5X SSC/0.1% Tween 20 solutions at 45°C.
2. Remove the rubber cement from the slide with forceps.
3. Remove the cover-slip by briefly soaking in 2X SSC at RT.
4. Wash the slide in two changes of 2X SSC/0.1% Tween 20 for 5 min at 45°C.
5. Wash the slide in two changes of 0.5X SSC/0.1% Tween 20 for 5 min at 45°C.
6. Briefly rinse the slide in dH₂O.
7. Air dry the slide out of direct light.
8. Apply 20 μ L of DAPI/Antifade solution to the hybridized area and cover with a coverslip (25x25 mm).

Microscope accessories

➤ Objectives

A 10X objective is suitable for scanning the target area. Higher magnification is required for signal analysis and should be performed with a 63X or a 100X oil immersion objective.

➤ Immersion oil

The immersion oil should be suitable for fluorescence microscopy.

➤ Lamp

A 100 watt mercury lamp with a maximum life of 200 hours is recommended. Replace the lamp before it exceeds 200 hours.

Filter Requirements

Fluorophore	Excitation max	Emission max
Green	496 nm	520 nm
Red	580 nm	603 nm
Aqua	431 nm	480 nm
DAPI	360 nm	460 nm

Signal visualization and interpretation

The signal should be visualized with an epi-fluorescence microscope equipped with the appropriate filters.

Procedure Note: The signals can be at different focal plane so it is important to focus up and down on the specimen to ensure that all the signals are counted.

- In normal diploid metaphase chromosomes and interphase nuclei, the Prenatal FISH Probe generates two

aqua, two green and two red signals corresponding to the two normal homologous chromosomes 13, 18 and 21, respectively.

- In cells with gain of the genomic region covered by any one of the probes, the number of signals of the corresponding colour will be increased.
- Partial trisomies, such as those caused by unbalanced derivatives of reciprocal translocations, will not be detected if the trisomy does not involve the specific regions covered by the **13/18/21 (RBI/ D18S478/ D21S1238)** FISH probe.

Recommendations and Limitations:

1. This product has been optimized for use on slides prepared from respective specimens according to routine cytogenetic methods. The manufacturer ensures that this product meets the analytical performance characteristics (sensitivity, specificity, reproducibility, and reportable range) established on intended tissues.
2. Each new lot of FISH Probe should be tested for locus specificity on a normal specimen, and on the intended tissue to verify proper reagent performance. It is the responsibility of the laboratory to establish the reportable ranges using positive and negative control specimens of the intended tissue.
3. Use of filters with spectral characteristics other than specified may adversely affect the strength of the signal. For example, the red fluorophore is visible through an orange filter, but the signals appear dim.
4. Metaphase FISH is recommended to characterize variant and atypical abnormal signal pattern.
5. The FISH assay is considered an adjunct to classical cytogenetics (karyotyping). The results of these assays must be interpreted in the full context of the patient's clinical history. A medical decision cannot be made based on the result of the FISH assay alone.